



**SOLID STATE SONICS AND ELECTRONICS, INC.**

4137 LOWER SILVER LAKE ROAD • TOPEKA, KANSAS 66618  
785-232-0497

FAX 785-232-0498

**\*\* FLASHER \*\***  
MODEL DF-100

**OPERATING INSTRUCTIONS**

Remove the FLASHER!@ from the sterile package.

Test the Defibrillator by placing the paddles (spoons) onto the two stainless steel electrodes . (one paddle on each electrode)

Discharge the defibrillator. (maximum 50 JOULES)

The orange spot will flash, showing that the defibrillator is functioning properly.

If the orange spot didn't flash a malfunction has occurred in the paddles or the defibrillator. Check system, replace defective equipment and retest.

@ FLASHER is a copyright item

08/01/2010